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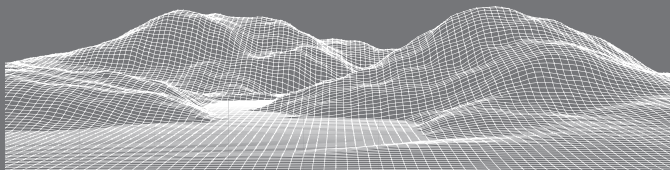
To ensure longevity of the new SD Card, please make sure to always shut down the printer properly. Meaning, always use the „shut down“ function in the start menu, wait approx. 30 seconds and then turn off the printer using the main switch on the back. Please do not turn off the printer by simply using the main switch, always do a proper shut down. Furthermore, please regularly delete old print jobs via the web interface from the printer.

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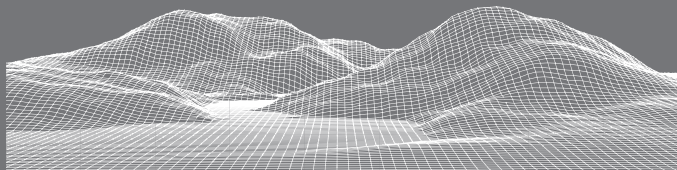
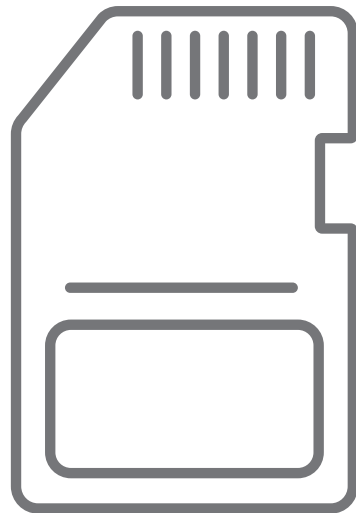
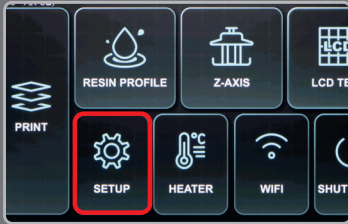
B

Instructions for changing the SD card

- 1 Please shut down your printer.
- 2 Remove the SD card from the back of the printer and replace it with the new one. Please take care to mount the SD card properly. You can hear a click sound once it's in the correct position.
- 3 Boot your printer.
- 4 Please click on „SETUP“. (A)

Click on “EXPAND SD“. (B)

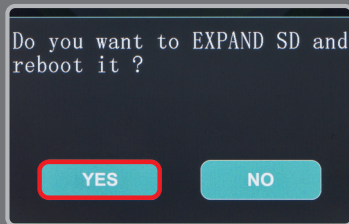
Click on „YES“ in the following dialogue. (C)

The printer will reboot now and is set up with the new SD card. (D)

- 5 Please make sure that the correct light power is set up in the printer. You can find the correct value for your printer on the service card that came with the printer.

To complete the process, the unit must be properly shut down and restarted.

C

D